



Application of Grip-Lok® 2200NUZA Securement Device

Prepare the skin according to the standard hospital protocol for dressing application. Skin must be CLEAN and DRY prior to Grip-Lok application. For full instructions, please consult the IFU at tidiproducts.com/grip-lok.



STEP 1
Open the top fabric strap and remove the inside release liner to expose the adhesive surface.



STEP 2
Slide the Grip-Lok under the catheter and center the hub above the adhesive area.



STEP 3
Secure the fabric over-strap on the catheter and press around and on top of the catheter and lumens to firmly attach the adhesive.



STEP 4
While holding the Grip-Lok and catheter in place with one hand, remove the bottom release liner from one side and adhere to skin, then repeat for the other side. Gently press down around the edges and on the surface of the Grip-Lok to ensure the adhesive connects to the skin and catheter.

